

Instructions for use

JEDX



JedX 3805 W FT HLV Respirator—Fish type with valve

Important

This respirator is intended to be used only once. Do not reuse it. Before use, the wearer must read and understand these User Instructions. Before entering the hazardous area make sure the respirator is fit correctly. Keep these instructions for reference.

Intended Use

JedX 3805 W FT HLV respirator is primarily for use in the healthcare sector, heavy industries and for consumers for every day life. This respirator is intended to protect the wearer against potentially hazardous and noxious particles in the air.

Description

The JedX JedX 3805 W FT HLV Particulate Respirator is classified as category III Personal Protective Equipment (PPE) impressed EC marking in conformity with European Commission Regulation (EU) 2016/425 regarding to PPE. It provides protection against particulates, mist, smoke, and oil-based aerosols. The packaging provides the mask with hygienic protection prior to its use. This product does not contain components made from natural rubber latex.

This respirator has a filter efficiency level of at least 99% against particulate aerosols free of oil per European EN 149:2001+A1:2009 standard.

Warnings



- Always be sure that the respirator is suitable for the application, fitted correctly and worn during all periods of exposure. It should be replaced when necessary.
- Proper selection, training and use are essential for the product to help protect the wearer from certain airborne contaminants. Failure to follow all instructions and limitations and/or failure to properly wear the respirator during all periods of exposure may adversely affect the wearer's health, lead to severe or life-threatening illness or permanent disability.
- Before use, the wearer must be trained in use of the complete product in accordance with applicable Health and Safety standards/guidance.
- Not for use with beards or other facial hair or conditions that prevent a good seal between the face and the sealing edge of the respirator. This respirator does not protect against gases or vapors (i.e. anesthetic gases such as isoflurane or vapors from sterilants such as glutaraldehyde).
- Do not use in atmospheres containing less than 19.5% oxygen. Individual countries may apply their own limits on oxygen deficiency. Seek advice if in doubt.
- Do not use for respiratory protection against atmospheric contaminants/concentrations which are unknown or immediately dangerous to life and health (IDLH).
- This respirator was not designed to be worn by children.
- For suitability and proper use, follow local regulations, refer to all information supplied or contact an occupational hygienist, safety professional or JedX representative.
- Inspect your respirator before each use to ensure it is in good working condition. Examine all the respirator parts for signs of damage including the two ear loops/straps and the nose-clip. The respirator should be disposed of immediately upon observation of damage or missing parts.
- Leave the contaminated area immediately if breathing becomes difficult, and/or if dizziness or other distress occurs.
- Discard and replace the respirator if it becomes damaged, breathing resistance becomes excessive.
- Never alter, modify, wash, or repair this device.
- Maximum Operating Temperature: +50 degree Celsius.


Storage, Transportation & Shelf Life

Before use, store respirators in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture, and damaging chemicals.

When stored in original packaging between temperatures from -22°F (-30°C) to $+104^{\circ}\text{F}$ ($+40^{\circ}\text{C}$) and not exceeding 80% RH, the product may be used until the date specified on packaging located next to the "Use by Date" symbol.

Shelf Life of unopened product is 5 years from date of manufacture. Before initial use, always check that the product is within the stated shelf life.

Fitting instructions

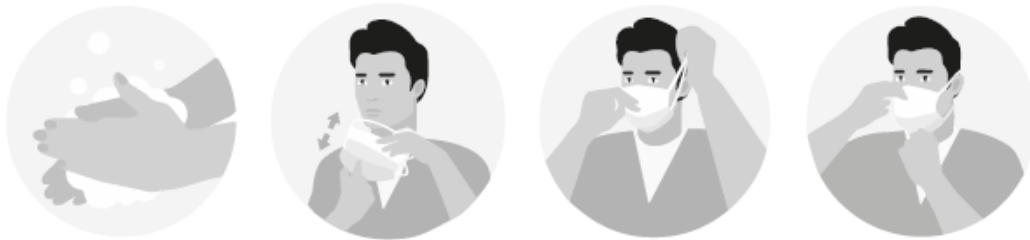

1. Remove the respirator from the package and place it on your palm with the nose piece at the lower end of the topmost part and the elastic bands hanging down.
2. Open the respirator and place it on your face so that the nose piece fits tightly onto your nose and the lower part onto your chin.
3. Hold the respirator in place and pull both elastic bands over your head.
4. Place the elastic bands so that the upper band sits on your head, above your ears and the lower band is on the back of your head below your ears.
5. Shape and press the nose piece in position using your both hands to ensure a tight fit.
6. The seal of the respirator on the face should be fit-checked before entering the workplace.

Fit Check

Cover the front of the respirator with both hands being careful not to disturb the fit of the respirator.

EXHALE sharply;

If air leaks around the nose, re-adjust the noseclip to eliminate leakage. Repeat the above fit check.

If air leaks at the respirator edges, work the straps back along the sides of the head to eliminate leakage. Repeat the above fit check.

If you CANNOT achieve a proper fit DO NOT enter the hazardous area. Users should be fit tested in accordance with national requirements.

Manufacturer

JedX Medcare

Köysikuja 1

01640 Vantaa

FINLAND

www.jedx.fi

